

DEC27 MULTIGENIUS ELECTRO STIMULATION




MULTIGENIUS DEVICE

- Multigenius is a new concept in microcurrent treatments.
- This machine incorporates Microcurrent and Kotz current to achieve the maximum effects on:
 - body conditioning;
 - body lifting;
 - muscles strengthening;
 - cellulite improvement;
 - face lifting.

TECHNICAL SPECIFICATION





- Power supply: 230 V 50-60 Hz
- Overall dimensions: 60x41x20 cm
- Weight: 7 kg
- Fuses: 2 of 500 mA
- Power consumption: 35 Watt
- Output frequency: 1 – 2500 Hz
- Output power on 1k Ohm: 5 mA
- Wave shape of output: Sinusoidal/Square

Multigenius supplied accessories

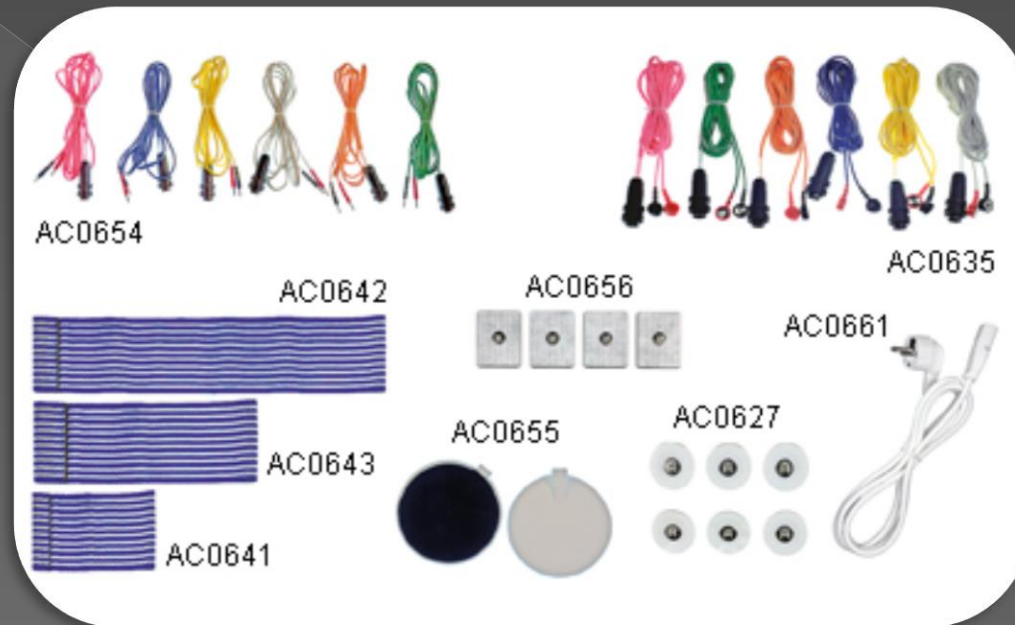


- AC0642 – 2 x 1 metre elastic straps (large)
- AC0643 – 4 x 60 cm elastic straps (medium)
- AC0641 – 4 x 40 cm elastic straps (small)
- AC0654 – 6 coloured cables for traditional rubber pads
- AC0655 – 12 traditional rubber pads
- AC0635 – 6 coloured cables for disposable pads
- AC0627 – 24 disposable face pads
- AC0656 – 12 disposable body pads
- AC0661 – 1 x mains lead with plug

Multigenius body treatment

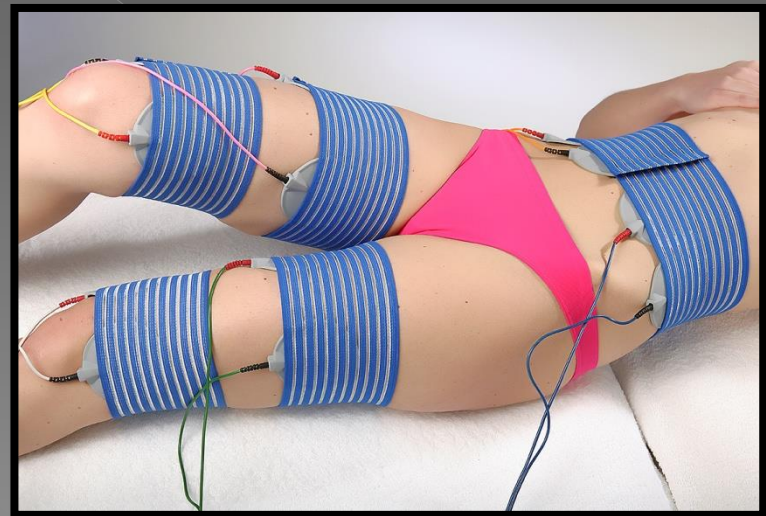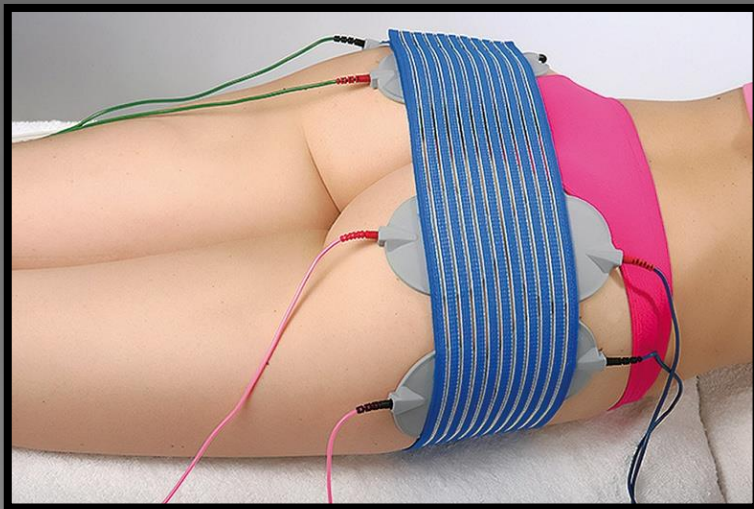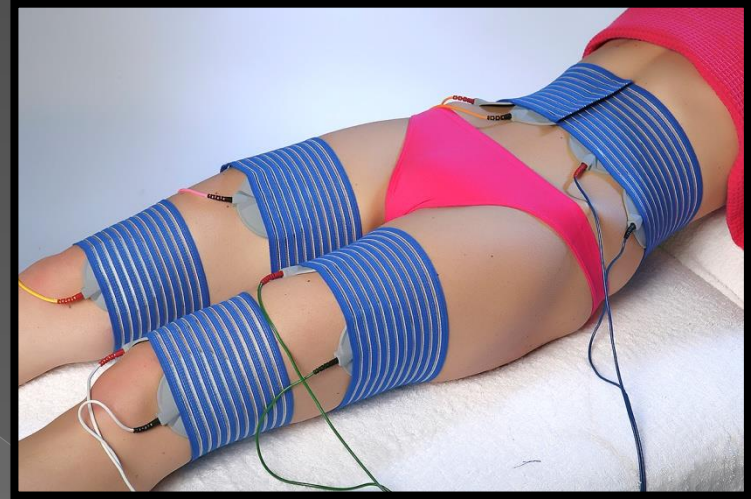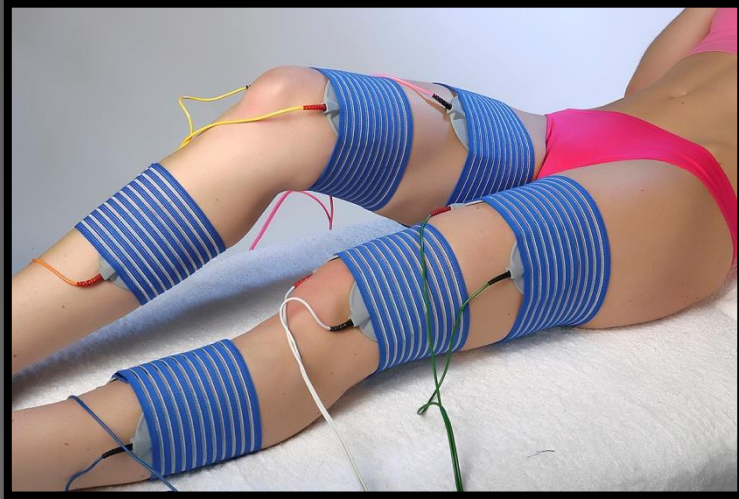

Multigenius body treatment

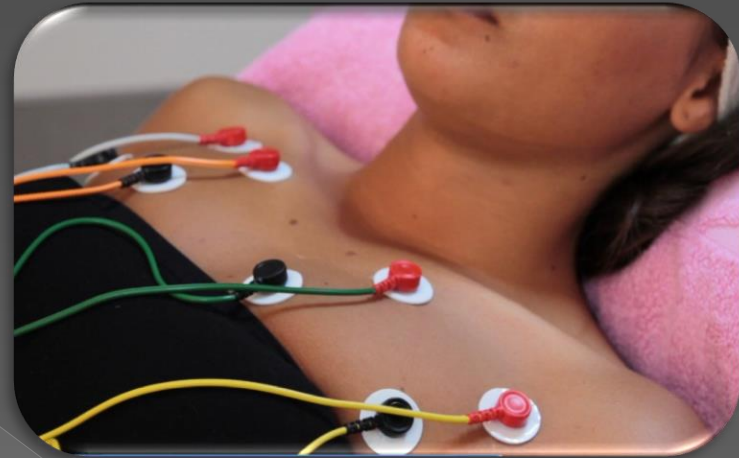

How Multigenius works on the muscles and body

The Multigenius offers a combination of two electrical currents in two different wave forms:



MULTIGENIUS MICROCURRENT

Microcurrent originates from a traditional Faradic current and it is applied in microamps (1/1000 of a milliamp) whereas traditional Faradic current is applied in milliamps (1/1000 of an amp).

MICROCURRENT

=

A MODIFIED FARADIC CURRENT APPLIED
AT A LOW INTENSITY

MUTIGENIUS MICROCURRENT

- The low intensity of microcurrent is absorbed into the cells and stimulates the functions of the cells.
- All the cells are boosted thus making the cell healthier. Consequently:
 - Healthy cells make healthy tissues,
 - Cells are stronger to fight free radicals,
 - Excess toxins trapped between cells are eliminated

MULTIGENIUS MICROCURRENT

- Healthy cells take up less space by creating a denser and a stronger tissue:

the skin and the muscles consist of these cells.

- Microcurrent is not irritating to the nerve endings therefore only a slight pulsing is felt by the client; a muscle contraction is not visible.

MULTIGENIUS KOTZ CURRENT

- Kotz current is generated at a very high frequency.
- It is an alternating current of up to 2500 Hz (cycles per second)
- The skin has a very low resistance to this current and therefore **Kotz current** is able to produce a **deep action in the muscles**.

MULTIGENIUS KOTZ CURRENT

- Kotz current has an excellent effect on increasing metabolism of the cells of the skin and muscles
- Kotz current increases the blood and lymphatic circulation in the area being treated in order :
 - ❖ to increase the level of nutrients brought into the cells;
 - ❖ to help in removing the residue of Lactic acid from the cells.
- Kotz current is very effective for muscle strengthening

MULTIGENIUS CURRENTS SUMMARY

MICROCURRENT:

- boosts cell metabolism;
- helps to eliminate cellulite;
- gives a lifting, countouring effect to the face and body.

MULTIGENIUS CURRENTS SUMMARY


KOTZ CURRENT

- increases cell metabolism, blood and lymphatic circulation;
- is responsible for muscle strengthening and body conditioning;
- helps to eliminate cellulite.

MULTIGENIUS CURRENTS SUMMARY


- MICROCURRENT and KOTZ CURRENT are used both independently and together throughout the programmes to produce a variety of effects

MICROCURRENT WAVE FORM

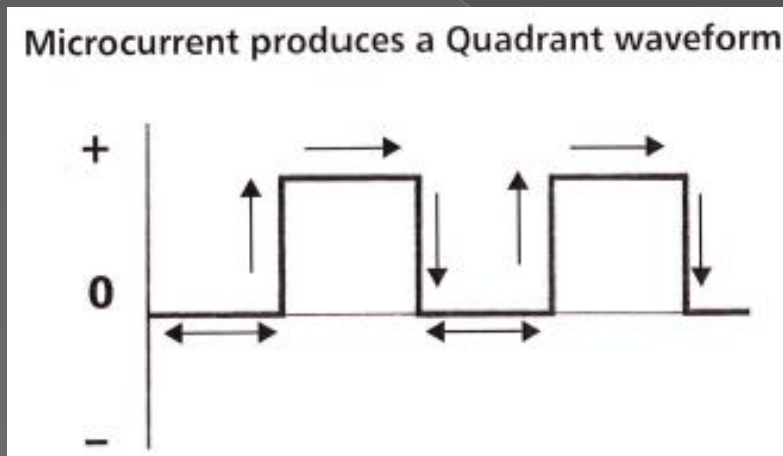

- This wave form is constant for a longer period than the Kotz Sine wave form.
- The muscles fibres are held in a degree of tension for a short period of time at the highest point of intensity.
- This gives the microcurrent time to pass into the cells without stimulating the sensory nerve endings.

KOTZ CURRENT WAVE FORM

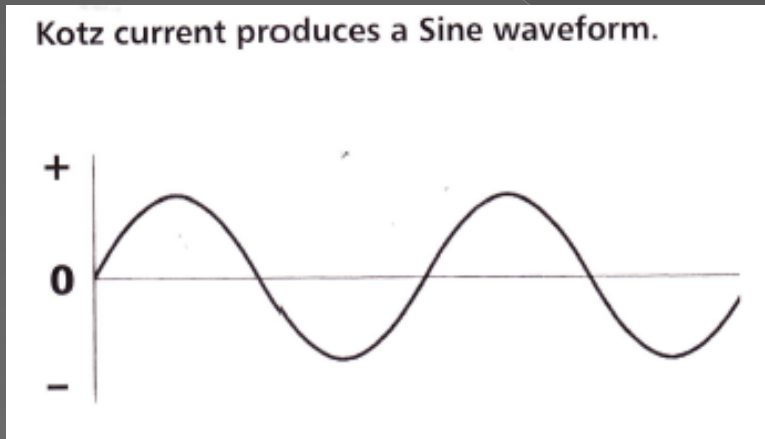

- This waveform rises and falls more gradually than the Quadrant waveform.
- Here the current slowly rises and as soon as it reaches its peak it begins to fall.
- This causes a deep action with very little sensation to the client.

MULTIGENIUS OFFERS THE THERAPIST :

- ◉ User friendly equipment;
- ◉ A multiple of body programmes to ensure each client is treated individually for their specific needs;
- ◉ Face lifting programme;
- ◉ Timer facility;
- ◉ Two methods of application to choose from traditional rubber electrodes or disposable stick on pads;
- ◉ An effective treatment, results are evident in a short space of time depending on the condition being treated;
- ◉ Increased business opportunities in the salon.

MULTIGENIUS OFFERS THE CLIENT:

- ◉ a very comfortable treatment, the client will feel a slight pulsing in the muscles (there is not a visible contraction of the muscles);
- ◉ results will show in short space of time, especially when a course of treatments are followed;
- ◉ motivation and encouragement. The effectiveness of the programmes will delight the clients;
- ◉ the combination of both Microcurrent and Kotz currents offer optimum results to improve the condition and the strenght of the tissues and muscles;
- ◉ a visual improvement in the appearance of the skin and body contour.

THANKS FOR YOUR KIND INTEREST IN DECOMEDICAL PRODUCTS

